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
**HK Pregnancy Tester  
for Horses  
Operating Instructions**

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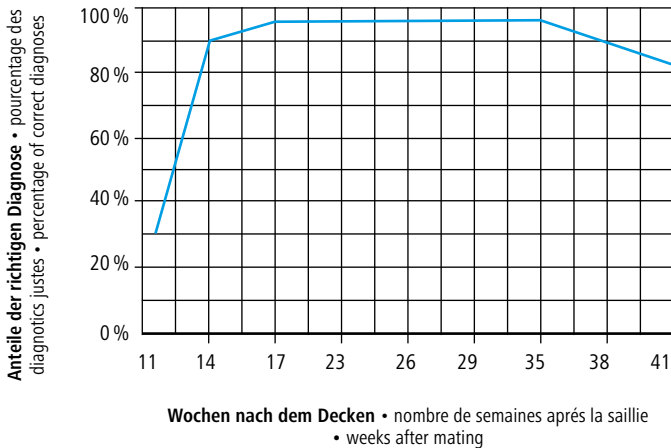Figure 1: The probe should be placed in the stifle fold in front of and above the udder on the abdominal wall and directed to the animals opposite side, between the hip-bone and the rear-rip

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Results of a pregnancy diagnostics test from mares between the 11<sup>th</sup> and 41<sup>th</sup> week after mating.

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Diagnosis spot and direction of the probe of a 12 week minimum pregnant mare.

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# **EN** HK pregnancy diagnosis unit for horses

With growth of the fetus and increase of uterine fluid, the uterus considerably enlarges as pregnancy progresses. In the 3rd to 4th month it will lower into the abdominal cavity. This is the best time for using the HK pregnancy detector. Diagnosis is possible from the 80th day, although this is affected by animal size, but is optimal from the 90th day. A clear audible continuous tone indicates a pregnant animal.

## **1. Operating-test**

After activating the power switch on upper panel of the detector, apply some contact-oil on the face of the probe. Next, place the metal test probe against the probe face. If this produces a clear continuous tone and if the pregnancy control lamp lights up red continuously, the pregnancy tester is in working order and ready for use. Pay attention to the battery power control. If the red light is on, the battery must be changed very soon. The battery is completely empty if no light is on and must be changed immediately.

## **2. Contact-oil**

The contact oil ensures airless contact between the animal's skin and the face of the probe. Before each diagnosis, apply a liberal amount of oil on the face of the probe. Any clean oil or gel, with exception of mineral oil, is suitable for this purpose. Paraffin oil, for example, has good properties. Do not handle with any used oil.

## **3. Diagnosis-spot and scanning-direction of probe**

Animals can be diagnosed from both sides. The optimal diagnosis spot is situated in the stifle fold in front of and above the udder on the abdominal wall. The probe has to be directed to the animal's opposite side at an angle of approx. 45°. By moving the probe slightly forward and backward in an angle of approx. 30°, the ultrasonic waves scan the abdominal region, including the uterus where uterine fluid will reflect the ultrasonic waves. The Figure 1 demonstrates the placement of the probe.

## **4. Diagnosis**

A clear audible continuous tone that lasts for 1-2 seconds indicates a pregnant animal. No signal, even after placing the probe more than once and using a good quantity of oil, indicates a non-pregnant animal.

## **5. Diagnosis precision**

Diagnosis precision is optimal from the 120th to the 280th day of pregnancy. However, the following aspects should be taken into consideration:

- The animal to be tested should stand in a natural position.
- If the probe is moved too much backwards, the ultrasonic waves can be reflected by the fluid of a full bladder or the hip-bones (pelvis) giving a false positive result.
- Apply fresh oil on the face of the probe for each new placing.
- In case of negative result, repeat testing at a later date. This gives additional certainty.
- Gain some experience of testing by using mares which are known to be pregnant or not pregnant respectively.

**SWITCH OFF POWER AFTER USE!**

## **6. Changing battery**

To change battery, open battery-compartment on the bottom of the tester. Therefore, push in the two lateral placed snap locks simultaneously with thumb and forefinger and take off the lid with the other hand. Use Alkaline batteries only (4 x 1,5 Volts Mignon). Other battery types diminish performance and should not be used.

## **7. Maintenance and care of detector**

After each use, probe and instrument should be cleaned carefully. The probe can be cleaned with warm soaped water, the instrument with a dry clean cloth only. The tester should be kept in a clean dry place. Do not drop or mistreat device, as this risks to damage the electronics and the probe. Careful maintenance and cleanliness will enhance the durability and performance of this instrument!

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## Warranty

Based on our General Terms and Conditions of Business, we offer a 2 years warranty for this device covering damage caused by material or manufacturing defects. This warranty begins on the date of purchase. The warranty does not cover:

1. Damaged caused by natural wear and tear (wear parts, sliding parts), excessive strain and incorrect use (non-compliance with operating instructions).
2. Damage caused by force majeure or sustained during transport.
3. Damage to rubber parts, e.g. on connecting lines. Warranty repairs are only carried out by the manufacturing plant or an authorised contracted workshop.

The manufacturer reserves the right to reject any warranty claims in the event of non-compliance. The device must for this purpose be returned to the point of sale, at the risk of the device owner, complete, packaged and with sufficient postage, including the original invoice (address see dealer stamp). Please provide as much information as possible on the nature of the problem. Costs incurred as a result of unjustified defect claims will be invoiced to the device's owner.

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## CE-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE mark confirms compliance with the Directives of the European Union. The declaration of conformity can be viewed by visiting [www.kerbl.com](http://www.kerbl.com)

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## Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.