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EN Fever thermometer for large animals

Please read the instructions carefully before using the thermometer for the first time, and keep these in a safe place.

Correct use

The fever thermometer for large animals is designed exclusively for use in the veterinary sector and not for use on hu-mans.

Operating Instructions

- Disinfect the probe before use.
- To switch it on, press the ON/OFF switch below the display. The last measured value appears and the letter „M“. As soon as the letters „Lo“ and a flashing „°C“ or „°F“ are displayed, the thermometer is ready for use.
- You can toggle the temperature from °C to °F and back again. To change the unit, press the ON/OFF button and hold it down until „Lo + °C“ or „Lo + °F“ appears on the display. You now have 2 seconds, while the „°C“ or „°F“ is not flashing, to press the ON/OFF button again, to change over the unit.
- As soon as a constant temperature is reached, the symbol „°C“ or „°F“ stops flashing. The highest measured temperature is shown in the display. Once measurement is complete, switch the thermometer off by pressing the ON/OFF button. Once a temperature has been displayed, the thermometer switches itself off automatically.

Memory function

Next time the device is used, the last value measured appears automatically in the LCD display with „°C“ or „°F“.

Rectal temperature measurement

This is the most accurate method from a medical point of view. The tip of the thermometer is inserted carefully into the rectum. The usual measuring time is around 8 seconds.

Cleaning and disinfection

The best way to clean the thermometer tip is by applying a disinfectant (e.g. 70% ethyl alcohol) with a damp cloth.

Safety precautions

- Do not allow the device to come into contact with hot water.
- Do not expose to high temperateres or direct sunlight.
- Do not drop the thermometer. It is neither shock-proof nor impact-resistant.
- Do not bend or open the device (except the battery compartment).
- Do not clean with thinners, petrol or benzene! Only clean with water or disinfectant.
- Do not immerse the Waterproof thermometers in liquid for longer than 30 minutes.
- The thermometer contains small parts (battery, battery compartment) which can be swallowed by children. For this reason, do not leave the thermometer unattended in the hands of children.
- Avoid bending the Non-Flexible thermometer tip. For the Flexible thermometer, avoid bending the tip by more than 45 degrees.
- If the ambient temperature is over 35 °C or 95 °C, dip the thermometer tip in cold water for approx. 5 to 10 seconds prior to measuring the temperature.
- Persistent fever requires medical treatment - please contact a veterinary surgeon!

Battery replacement

The battery is empty and needs replacing when the “▬” battery symbol appears on the right of the LCD. Remove the battery cover and replace it with a battery (preferably non-mercury) of the same type. Please note: The “+ ” sign up and “- ” sign down.

Technical data	
Measurement range:	32 ~ 43.9 °C / 90 ~ 111 °F
Measurement accuacy:	+/- 0,1 °C / 0,2 °F (35 ~ 41 °C / 95 ~ 102 °F)
Storage/transportation temperature:	-10 °C ~ +40 °C
Ambient temperature during use:	+10 °C ~ +35 °C
Battery type:	3.0VDC (CR2032)
Weight:	30 g

Calibration check

This thermometer is initially calibrated at the time of manufacture. If this thermometer is used according to the operation instruction, periodic re-adjustment is not required.

Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

UKCA-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE-/UKCA-mark confirms compliance with the Directives of the European Union or the relevant UK legislation.