**(GB)** Instructions for use/Technical description

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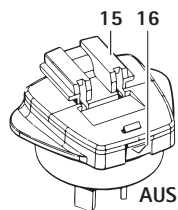
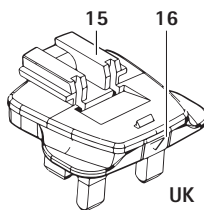
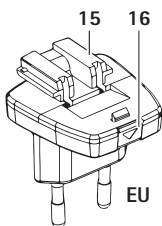
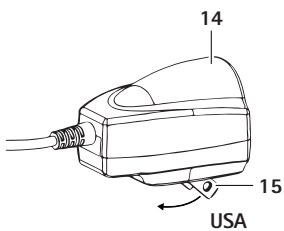
## Battery-powered clipping machine FAV5 CL/FORTIS

### Legend

- 1 Blade adapter
- 2 Battery-powered clipping machine FAV5 CL/FORTIS
- 3 Battery release lock
- 4 Charger
- 5 Mains power cord (with device plug)
- 6 Charging bay (for Li-ion battery)
- 7 Charge state indicator (for Li-ion battery)
- 8 Li-ion battery
- 9 Charge state indicator (for battery-powered clipping machine FAV5 CL/FORTIS)
- 10 Charging bay (for battery-powered clipping machine FAV5 CL/FORTIS)
- 11 On/Off switch
- 12 Blade

### Symbols on product

	Adhere to instructions for use
	Marking of electric and electronic devices according to directive 2012/19/EU (WEEE), see Chapter Disposal

### Contents

- 1. Safe handling ..... 3
- 2. Product description ..... 3
- 2.1 Scope of delivery ..... 3
- 2.2 Components required for operation ..... 4
- 2.3 Intended use of battery-powered clipping machine FAV5 CL/FORTIS ..... 4
- 2.4 Intended use of charger XT433/Li-ion battery XT431 ..... 4
- 2.5 Operating principle of the battery-powered clipping machine FAV5 CL/FORTIS ..... 4

- 2.6 Operating principle of charger XT433/Li-ion battery XT431 ..... 4
- 2.6.1 Charging principle ..... 4
- 2.6.2 Charging time ..... 5
- 3. Preparation and setup ..... 5
- 4. Working with the battery-powered clipping machine FAV5 CL/FORTIS, and the charger XT433/Li-ion battery XT431 ..... 6
- 4.1 System set-up ..... 6
- 4.1.1 Connecting the accessories..... 6
- 4.1.2 Attaching the blade..... 6
- 4.1.3 Attach the adapter to the power supply of the charger..... 7
- 4.1.4 Connecting the charger/Charging the Li-ion battery..... 7
- 4.2 Functional check of the battery-powered clipping machine FAV5 CL/FORTIS ..... 8
- 4.3 Function checks of the charger ..... 8
- 4.4 Safe handling blade ..... 8
- 4.4.1 Sharpening the cutting plate ..... 8
- 4.4.2 Changing the blade..... 8
- 4.4.3 Lubricating the blade..... 9
- 4.4.4 Changing the Li-ion battery ..... 9
- 5. Processing ..... 10
- 5.1 Cleaning/disinfection ..... 10
- 5.2 Control, care and inspection ..... 10
- 6. Maintenance ..... 11
- 7. Troubleshooting list ..... 11
- 7.1 Battery-powered clipping machine FAV5 CL/FORTIS ..... 11
- 7.2 Charger and Li-ion battery ..... 12
- 8. Technical Service ..... 13
- 9. Accessories/Spare parts ..... 13
- 10. Technical data ..... 14
- 10.1 Battery-powered clipping machine FAV5 CL/FORTIS ..... 14
- 10.2 Charger ..... 14
- 10.3 Li-ion battery ..... 14
- 10.4 Declaration of conformity ..... 14
- 11. Disposal ..... 14
- 11.1 For EU countries only ..... 14
- 11.2 Li-ion batteries ..... 15
- 12. United Kingdom ..... 15

- 13. Two year guarantee ..... 15
- 14. Distributor in the US/Contact in Canada for product information and complaints ..... 15

## 1. Safe handling

### CAUTION

**Federal law restricts this device to sale by or on order of a physician!**

**Risk of fatal injury from electric shock!**

- **Do not open the product (except for changing a charging bay).**
- **Connect the product only to a grounded power supply.**
- Prior to use, check that the product is in good working order.
- Store the charger and Li-ion batteries at room temperature.
- Keep dry the charger and Li-ion batteries.
- Charge the Li-ion batteries prior to first use.
- Observe our "Notes on Electromagnetic Compatibility (EMC)".
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
  - Use the product only according to these instructions for use.
  - Follow the safety and maintenance instructions.
  - Ensure the manufacturer's approval before using accessories that are not mentioned in the instructions for use.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.

- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

### Note

*The device can be used by children over the age of 8 as well as by persons with reduced physical, sensory or mental capacities or with lack of experience and knowledge, if these persons are supervised or have been instructed on how to use the device safely, and if they understand the risks associated with the use of the device.*

- Do not allow children to play with the device.
- Do not allow children to carry out cleaning and maintenance of the device without supervision.

## 2. Product description

### 2.1 Scope of delivery

Designation	Art. no.
Battery-powered clipping machine FAV5 CL/FORTIS	GT3xx-xx
Li-ion battery	XT431
Charger	XT433
Mains power cord (with device plug)	see Chapter Accessories/ Spare parts
Instructions for Use	TA016219

## Battery-powered clipping machine FAV5 CL/FORTIS

### Note

The blade GT330 #10 (USA+UK) for battery-powered clipping machine FAV5 CL/FORTIS is included in the scope of delivery. Various blades are available for different applications!

### 2.2 Components required for operation

- Battery-powered clipping machine FAV5 CL/FORTIS
- Blade
- Charger
- Mains power cord (with device plug)
- Li-ion battery

### 2.3 Intended use of battery-powered clipping machine FAV5 CL/FORTIS

#### Note

The device is intended exclusively for cutting animal hair.

The battery-powered clipping machine FAV5 CL/FORTIS is used for shearing small animals.

The battery powered clipping machine FAV5 CL/FORTIS is used with a blade to be selected according to the respective application.

### 2.4 Intended use of charger XT433/Li-ion battery XT431

The charger XT433 is used for charging the battery-powered clipping machine FAV5 CL/FORTIS and the Li-ion battery. There is one charging bay each for the two components.

### 2.5 Operating principle of the battery-powered clipping machine FAV5 CL/FORTIS

The battery-powered clipping machine FAV5 CL/FORTIS is switched on (off) by pushing the ON/OFF switch up (down).

### 2.6 Operating principle of charger XT433/Li-ion battery XT431

The charger XT433 is designed for the mains voltage/frequency range 100 V to 240 V / 50 Hz to 60 Hz.

To make the charger XT433 ready for operation, connect it to mains power by plugging in the mains plug.

The charger XT433 is equipped with two charging bays. The first charging bay is for charging the battery-powered clipping machine FAV5 CL/FORTIS, while in the second charging bay the Li-ion battery is charged.

Each charging bay has an indicator field on the charger front panel assigned to it.

Charging starts automatically, as soon as a Li-ion battery is inserted in the charging station.

The charging time in the charging bays depends on the charge state and capacity of the Li-ion batteries.

#### 2.6.1 Charging principle

The Li-ion batteries are charged by constant current pulses.

The charge state of the Li-ion battery is monitored during charging. Monitoring of the charging curve ensures that the battery is fully charged without overcharging.

Additionally, the battery temperature and charging time are monitored as well.

Only one charging bay is active at a time. The machine (front charging bay) is always charged first. It is then first examined whether a battery is inserted in the rear charging bay. If this is the case, the battery is charged. If a battery is inserted into the rear charging bay while a machine is being charged in the front charging bay, the LED for the rear charging bay blinks green once (battery detected) and then goes out until the battery of the machine in the front charging bay is charged.

Only then does the charging process for the battery in the rear charging bay start.

The charger monitors the battery status and indicates the charge state via two LED indicators.

LED indicators:

- Blinking green: battery charging
  - Illuminated green = battery is completely charged
  - Blinking red = battery or charging station defective
- Have the manufacturer repair the product.

### 2.6.2 Charging time

Charging is stopped as soon as the maximum charging time is reached.

The charging time is approx. 240 min.

## 3. Preparation and setup

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- When setting up and operating the product, adhere to
- national regulations for installation and operation,
  - national regulations on fire and explosion protection.
  - operating advisories according to IEC-/VDE regulations.

**DANGER**

**Fire and explosion hazard!**

- Do not operate the product in explosion-hazard areas.

**CAUTION**

**Malfunction of, or damage to the product caused by improper setup!**

- Do not expose the product to direct sunlight or moisture.
- Do not cover the ventilation slots of the product.
- Connect the product to the specified voltage of 100 V to 240 V/50 Hz to 60 Hz.
- Ensure that the mains power socket used for supplying the product is freely accessible.

## Battery-powered clipping machine FAV5 CL/FORTIS

### 4. Working with the battery-powered clipping machine FAV5 CL/FORTIS, and the charger XT433/Li-ion battery XT431

#### 4.1 System set-up

##### 4.1.1 Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

##### 4.1.2 Attaching the blade

**WARNING**

**Risk of injury caused by sharp cutting plates!**

- Handle the blade of clipping machine FAV5 CL/FORTIS with appropriate caution.

#### Note

The blade GT330 #10 (USA+UK) for battery-powered clipping machine FAV5 CL/FORTIS is included in the scope of delivery. Various blades are available for different applications!

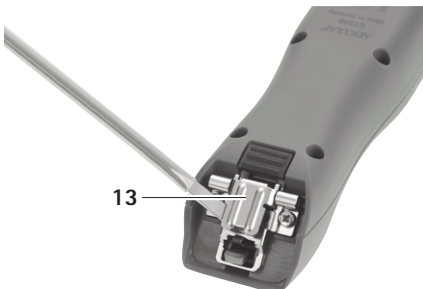
Fig. 1

- If the knife hinge is closed: Open knife hinge 13 with a screwdriver, see Fig. 1.

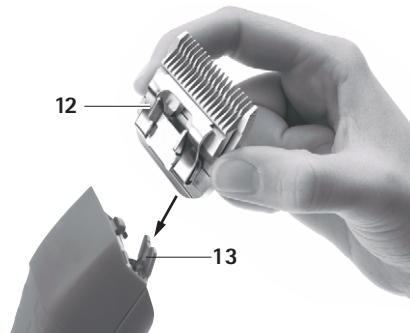
Fig. 2

- Install blade 12 on open knife hinge 13, see Fig. 2.
- Press ON/OFF switch 11.
- Press blade 12 gently in the direction of the clipping machine FAV5 CL/FORTIS, see Fig. 2. The blade 12 engages with an audible click.
- Ensure that the blade 12 is fitted correctly on the clipping machine FAV5 CL/FORTIS.

### 4.1.3 Attach the adapter to the power supply of the charger

The adapter for EU countries is installed on the power supply of the charger at the factory.

*Note*

*The USA adapter is integrated into the power supply. The USA adapter can be folded out if no other adapter is installed on the power supply.*

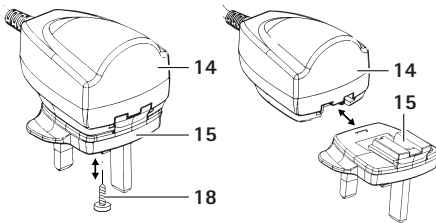
Fig. 3

#### Remove EU/UK/AUS from the power supply.

- Loosen the fixation screw 18 in adapter 15, see Fig. .
- Press the lock release 16 and remove the adapter 15 from the power supply.

#### Install the EU/UK/AUS adapter on the power supply

- Push the adapter 15 onto the power supply 14 until the lock 16 clicks into place, see Fig. .
- Screw the fixation screw 18. Screw the fixation screw 15.

### 4.1.4 Connecting the charger/Charging the Li-ion battery

**WARNING**

**Fire hazard due to battery poles shorted by fluids or metal components!**

- Do not short the Li-ion battery.

**WARNING**

**Risk of injury and material damage due to damaged Li-ion batteries!**

- Prior to charging, check the Li-ion batteries for any damage.
- Do not use or charge damaged Li-ion batteries.

**CAUTION**

**Loss of capacity/performance due to prolonged storage of discharged Li-ion batteries!**

- Store Li-ion batteries only in fully charged condition and recharge stored batteries once every month.

**CAUTION**

**Destruction of the product, charging bay or Li-ion battery when inserting the wrong type of battery in the charging bay!**

- Insert Li-ion batteries only in the appropriate charging bay.
- Insert the Li-ion battery in the correct position.

*Note*

*Switch off the battery-powered clipping machine FAV5 CL/FORTIS before charging!*

*Note*

*Li-ion batteries heat up slightly during charging.*

*Note*

*Unplug the mains power cord at the mains socket when the charger is not going to be used!*

- To connect charger 4: Plug in the device plug of mains power cord 5 in charger 4.
- Plug in the mains plug of mains power cord 5 at the mains power socket.

## Battery-powered clipping machine FAV5 CL/FORTIS

- Place Li-ion battery **8** in charging bay **6** of the charger **4** and charge.
  - or -
- Place battery-powered clipping machine FAV5 CL/FORTIS **2** in charging bay **10** of the charger **4** and charge.
 

Charge state indicator **7** or **9** blinks green.

After charging, the charge state indicator **7/9** shines green and the battery-powered clipping machine FAV5 CL/FORTIS **2**/Li-ion battery **8** is ready for operation.
- Place Li-ion battery **8** in charging bay **6** of the charger **4** and charge.
  - or -
- Place battery-powered clipping machine FAV5 CL/FORTIS **2** in charging bay **10** of the charger **4** and charge.
 

Charge state indicator **7** or **9** blinks green.

After charging, the charge state indicator **7/9** shines green and the battery-powered clipping machine FAV5 CL/FORTIS **2**/Li-ion battery **8** is ready for operation.

The product is ready for operation.

### 4.2 Functional check of the battery-powered clipping machine FAV5 CL/FORTIS

- Carry out a visual inspection.
- Check the switch position.
- Install the battery, see Chapter Changing the Li-ion battery.
- Mount the blade correctly and lubricate it if necessary, see Chapter Safe handling blade.
- Keep a charged Li-ion battery on hand, as reserve.
- Clean battery-powered clipping machine FAV5 CL/FORTIS prior to use, see Chapter Cleaning/disinfection.

### 4.3 Function checks of the charger

- Carry out a visual inspection.
- Prior to connecting the product to mains power:
  - Check mains power cord **5** for possible damage.
  - Check the product for any damage (e.g. bent contacts of the charging bays).
- Plug in mains power cord **5** at the device socket of the charging station.
- Plug in the mains plug at the mains power socket.

### 4.4 Safe handling blade

#### 4.4.1 Sharpening the cutting plate

*Note*

*For sharpening the cutting plate, please contact Technical Service, see Chapter Technical Service.*

#### 4.4.2 Changing the blade

**Risk of injury caused by sharp cutting plates!**

- **Handle the blade of clipping machine FAV5 CL/FORTIS with appropriate caution.**

Fig. 4

- Press ON/OFF switch 11 to switch off battery-powered clipping machine FAV5 CL/FORTIS.
- Press blade lock 14, see Fig. 4.

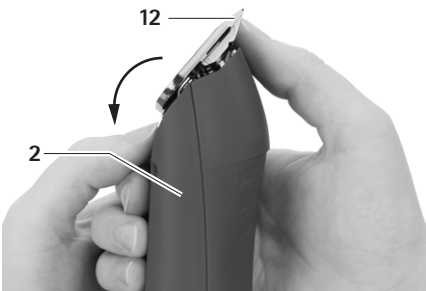
Fig. 5

- Fold forward blade 12, see Fig. 5.
- Detach blade 12.
- Clean open knife hinge with a brush.
- Install blade 12 on knife hinge.
- Press ON/OFF switch 11.

- Slightly push blade 12 towards battery-powered clipping machine FAV5 CL/FORTIS. Blade 12 will click into position.
- Check that blade 12 is correctly seated on battery-powered clipping machine FAV5 CL/FORTIS.

#### 4.4.3 Lubricating the blade

- Clean the upper and lower cutting plates with a soft cloth or brush. Make certain that the gaps between the cutting teeth are cleaned properly.
- Slightly lubricate the upper and lower cutting plates with special lubricant for blades GT604.

#### 4.4.4 Changing the Li-ion battery

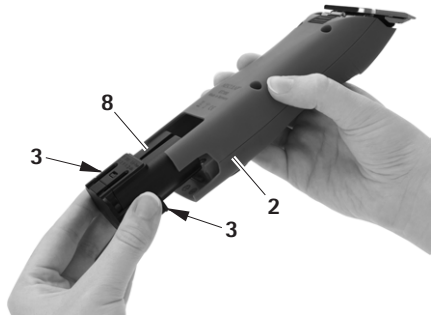
Fig. 6

- Hold battery-powered clipping machine FAV5 CL/FORTIS 2 and press battery release lock 3.
- Pull out Li-ion battery 8 from battery-powered clipping machine FAV5 CL/FORTIS 2.
- Insert the charged Li-ion battery 8 in battery-powered clipping machine FAV5 CL/FORTIS 2 as far as it will go. The battery-powered clipping machine FAV5 CL/FORTIS 2 is ready for operation.

## Battery-powered clipping machine FAV5 CL/FORTIS

### 5. Processing

**CAUTION**

**Damage to, or destruction of the product caused by mechanical cleaning/disinfecting or sterilization!**

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

#### 5.1 Cleaning/disinfection

*Note*

*Always follow disposal and hygiene instructions and regulations!*

##### Manual cleaning/disinfection

**DANGER**

**Risk of electric shock and fire hazard!**

- Unplug the device before cleaning.
- Ensure that no fluids will penetrate the product.
- Leave the product exposed to air for at least 1 minute after cleaning/disinfecting.

**CAUTION**

**Damage to the product due to incorrect cleaning or inappropriate cleaning/disinfecting agents!**

- Use any commercial detergent for surface cleaning.
- Never immerse the product in water or detergent.
- Do not introduce any fluid into the charging bays.
- Clean the contacts in the charging bays, applying extreme care.

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- If necessary, repeat the cleaning/disinfecting process.

#### 5.2 Control, care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- Lubricate blade if required, see Chapter Lubricating the blade.
- Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- Inspect the blade for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

## 6. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product at least once a year.

### Service addresses

In the UK  
KG PRODUCTS  
243-251 City Road, Fenton  
Stoke-on-Trent, Staffs  
ST4 2PX  
United Kingdom,  
Phone (44) - 01782 844 866

DIAMOND EDGE  
126 Gloucester Rd.  
Brighton, BN1 4BU.  
United Kingdom  
Phone (44) - 01273 605 922

Or in the US  
A & J Sales and Service  
42 - A Frame Drive  
West Burke, VT 05871  
USA  
Phone: (800) 447-1957

Or in Germany  
Albert Kerbl GmbH  
Felizenzell 9  
84428 Buchbach / Germany  
Phone: +49 (0) 8086 933-100  
Fax: +49 (0) 8086 933-500  
E-Mail: [info@kerbl.com](mailto:info@kerbl.com)  
[www.kerbl.com](http://www.kerbl.com)

Other service addresses can be obtained from the address indicated above.

## 7. Troubleshooting list

### 7.1 Battery-powered clipping machine FAV5 CL/FORTIS

Malfunction	Detection	Cause	Remedy
Battery-powered clipping machine FAV5 CL/FORTIS does not start	Check switch position	Li-ion battery not loaded	Load Li-ion battery
	-	Li-ion battery defect	Insert new Li-ion battery
	-	Li-ion battery not inserted	Insert new Li-ion battery
Battery-powered clipping machine FAV5 CL/FORTIS has come to a stop	-	Li-ion battery deactivated through protective circuit	Deactivate and reactivate the control unit
	-	Li-ion battery is empty	Insert new Li-ion battery
Battery-powered clipping machine FAV5 CL/FORTIS does not load into the charging station	LED indicators do not light up	-	see Chapter Charger and Li-ion battery

## Battery-powered clipping machine FAV5 CL/FORTIS

### 7.2 Charger and Li-ion battery

Malfunction	Detection	Cause	Remedy
Charger not working	LED indicators do not light up	Mains power cord not plugged in	Plug mains power cord into the device socket on the charger and into the socket of the mains power supply.
	Li-ion battery not loaded	Mains power cord defect	Replace mains power cord
		Charger defect	Send product to manufacturer for repair
Li-ion battery cannot be loaded	Li-ion battery in place, charge state indicator does not light up	Charger contacts dirty or damaged	Clean contacts in the charging bay, see Chapter Cleaning/disinfection or send to manufacturer for repair
		Li-ion battery defect	Send Li-ion battery to manufacturer for repair
		Charger defect	Send charger to manufacturer for repair
		Excess temperature is detected in the Li-ion battery during charging	Remove Li-ion battery from charging bay, leave to cool and start the charging process again. If the problem occurs again, send battery to manufacturer for repair
		Charging fault or Li-ion battery defect	Remove Li-ion battery from charging bay and repeat charging process, if necessary in another charging bay. If the problem occurs again send Li-ion battery to manufacturer for repair

## 8. Technical Service

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

### Service addresses

In the UK

KG PRODUCTS

243-251 City Road, Fenton

Stoke-on-Trent, Staffs

ST4 2PX

United Kingdom,

Phone (44) - 01782 844 866

DIAMOND EDGE

126 Gloucester Rd.

Brighton, BN1 4BU.

United Kingdom

Phone (44) - 01273 605 922

Or in the US

A & J Sales and Service

42 - A Frame Drive

West Burke, VT 05871

USA

Phone: (800) 447-1957

Or in Germany

Albert Kerbl GmbH

Felizenzell 9

84428 Buchbach / Germany

Phone: +49 (0) 8086 933-100

Fax: +49 (0) 8086 933-500

E-Mail: [info@kerbl.com](mailto:info@kerbl.com)

[www.kerbl.com](http://www.kerbl.com)

Other service addresses can be obtained from the address indicated above.

## 9. Accessories/Spare parts

Art. no.	Design	Li-ion battery	Charging station	Accessories	Art. no.	Design
GT30x-xx	Europe, Great Britain, USA, Australia	XT431	XT433	GT604800 Lubricator bottle	XT434-xx / XT436-xx	Europe, Great Britain, USA, Australia

## Battery-powered clipping machine FAV5 CL/FORTIS

### 10. Technical data

#### 10.1 Battery-powered clipping machine FAV5 CL/FORTIS

Model	GT3xx-xx
Stroke frequency	2 300/min
Charging voltage/Output voltage	max. 8.4 V
Charging current/Output current	max. 1.2 A
Weight (without blade and battery)	385 g
Weight (with blade and battery)	450 g
EN 60335-1, EN 60335-2-29	

#### 10.2 Charger

Model	XT433
Mains voltage ranges (Current intake)	100–240 V
Frequency	50–60 Hz
Charging voltage/Output voltage	max. 7.4 V
Charging current/Output current	max. 1 A
Weight including power supply	242 g
Quality marks	CE, UL

#### 10.3 Li-ion battery

Model	XT431
Cell type	Li
DC voltage	7.4 V
Capacity	approx. 2,850 mAh
Charging time	approx. 240 min

#### 10.4 Declaration of conformity

	<p>We declare under our sole responsibility that this product complies with the following standards and regulatory documents:</p> <p>2004/108/EC EMC guidelines:          DIN EN 60335-1          DIN EN 60335-2-8</p> <p>Safety of household and similar electrical appliances          - general requirements          - for hair clipping machines</p> <p>Aesculap Suhl GmbH</p>
---	---

### 11. Disposal

Electrical tools, accessories and packaging should be sent for environmentally responsible recycling.

#### 11.1 For EU countries only

	<p>Do not dispose of electrical tools in domestic waste!</p> <p>According to European Directive 2012/19/EU on old electrical and electronic appliances and its implementation in national law, electrical tools that are no longer suitable for use must be separately collected and sent for environmentally responsible recycling.</p>
---	--

## 11.2 Li-ion batteries

Do not dispose of batteries in domestic waste, fire or water. Defective or used batteries must be collected, recycled or disposed of in an environmentally-friendly way according to Directive 2006/66/EG.

## 12. United Kingdom

Kerbl UK Ltd.  
8 Lands End Way  
Oakham, Rutland  
LE15 6RF UK  
[www.kerbl.co.uk](http://www.kerbl.co.uk)  
Phone: 0044 01572 722558

## 13. Two year guarantee

Dear Customer,

Thank you for choosing a product from our company. For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices, using high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

## 14. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.  
3773 Corporate Parkway  
Center Valley, PA 18034  
USA

### Service addresses

A & J Sales and Service  
42 - A Frame Drive  
West Burke, VT 05871  
USA

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**Aesculap Schermaschinen GmbH**