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## EN Operating instructions AniShock Comfy Akku Art. No. 11267

### Safety instructions

Read the following instructions carefully before using the device.

### CAUTION! High voltage! Danger of injury and material damage!

- Do not touch the electrodes. Even with the device switched off, a voltage impulse can still be produced (residual charge).
- Never use the device on humans.
- Keep out of the reach of children.
- Switch the device off before cleaning it and ensure any residual charge in the electrodes has dissipated.
- Do not operate the device in liquids.
- Do not use the device to strike the animals.

### CAUTION! Risk of physical injury and material damage if used incorrectly!

- Do not expose the device to excessive temperatures, sunlight or fire.
- Protect the device from vibration and shocks.
- Only charge the device with the power pack provided with the original device.
- Maintain constant supervision when charging. Do not leave the battery to charge unattended and do not leave the charger plugged in overnight
- Only start a charging process when the green LED on the device has gone out.
- Interrupt the charging process if the green LED on the charging cable lights up.
- Immediately interrupt charging if you notice the battery is heating up excessively
- Do not use the device if the casing is broken

### Correct use

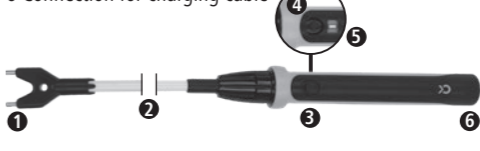
The device is used as a droving implement for adult cattle and pigs. Any other use, including use on humans, shall be deemed as not being compliant with the intended use. Animal protection regulations applicable in each country must also be observed.

### Function

The device generates high voltage and delivers an electric shock when the electrodes touch the animal.

### Setup

- 1 Electrodes
- 2 Polycarbonate shaft
- 3 Operating button
- 4 On/Off switch
- 5 LED displays
- 6 Connection for charging cable

### LED displays

1) Operating state LED  
red = Device switched on;  
LED off = Device switched off

2) Battery charge state LED  
green = Battery is sufficiently charged;  
LED off = Battery must be charged

3) Current surge LED  
red = Device is generating current surge;  
LED off = Device is not generating current surge

### Charging the batteries

1. Switch the device off (OFF).
2. Connect the charging cable to the device (6).
3. Connect the charging cable to the USB socket or via the supplied mains adapter to the mains supply (230 V).

First charge: 6 hours

Each further charge: 4 hours

LED on the charging cable:

Green = The battery is full,  
Red = The battery is charging

### Operation

1. Switch the device on.
2. Press the operating button (3) and at the same time press the electrodes (1) onto the animal's hind leg muscles. The device delivers a current surge of max. one second duration. An audible signal will be heard for the duration of the shock.
3. Take the device off the animal.
4. Switch the device off.
5. Always recharge batteries immediately since there is a risk of deep discharge and battery failure if left discharged for long periods.

### Cleaning

#### CAUTION! High voltage!

Before cleaning the device, remove the batteries and allow any residual charge to dissipate. The device can be cleaned with conventional cleaning agents and water.

### Disposal

The device must not be disposed of in household waste. Under the terms of the EU Directive on the disposal of old electrical and electronic equipment, the device can be taken to local authority-run refuse sites or recycling plants free of charge, or returned to dealers who offer a returns service. Correct disposal protects the environment and prevents any possible harmful impacts on humans and the environment.

### Technical data

max. peak voltage	6000 V
Supply voltage	100-240 V AC, 50-60 Hz
Safety class	IP 66
Length	0.93 m

### Spare parts

Art. No.	Designation
11268	Spare shaft, 71 cm
11269	Spare shaft, 83 cm

### Declaration of conformity

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations of Directive 2014/35/EU. The CE mark confirms compliance with the Directives of the European Union. The declaration of conformity can be viewed by visiting [www.kerbl.com](http://www.kerbl.com)

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Albert Kerbl GmbH Felizenzell 9
Kerbl UK Ltd 8 Lands End Way